# FIRST CHECK®

**IMPORTANT NOTE:**

When reading your result, a line appearing next to the word Drug represents a **Negative Result**. **NO** line appearing next to the word Drug represents a **Preliminary Positive Result**.

## ABOUT MARIJUANA (THC)

The *First Check® Marijuana Home Drug Test* is a highly accurate test that detects the presence of marijuana (THC) in urine.

Marijuana (also known as grass, pot, reefer, weed) is the mind-altering substance made of the dried leaves and flowers of the hemp (cannabis) plant. It can be hand-rolled into cigarettes called "joints," stuffed into hollowed-out cigars called "blunts" or put into small pipes and smoked. Marijuana can also be mixed into food or brewed into tea. The primary psychoactive substance in marijuana, known as delta-9-tetrahydrocannabinol (THC), can be detected in urine approximately 2 hours after use and up to 40 days (or longer) depending on use. Age, weight, diet, exercise and metabolism also affect detection time. Unlike most other drugs, THC is not water-soluble and is stored in the fat cells of the body, contributing to its longer detection period.

## BEFORE YOU TEST

Read these instructions completely.

- Have a watch, clock or timer available.
- Check expiration date on the box. Do not use if product has expired.

- In order to test in your home, you will only need these items:
  - Test Lid (in foil pouch)
  - Collection Cup
  - Instruction Insert

- Set aside these items; they will only be needed if you choose to ship the sample for confidential confirmatory lab testing:
  - Pre-addressed Mailing Box
  - Plastic Transportation Bag
  - Identification Label

## Step 1 Take the Test

- When you are ready to test, remove the Test Lid from the sealed foil pouch. Remove the plastic wrapper from the Collection Cup. Fill the cup with urine to at least the Minimum Urine Level line.


## Step 3 Read Your Result

- You can read your result after 5 minutes, but do not wait longer than 10 minutes.
- A red or pink colored line must appear next to the word Control to indicate the test worked correctly.

First Check Lid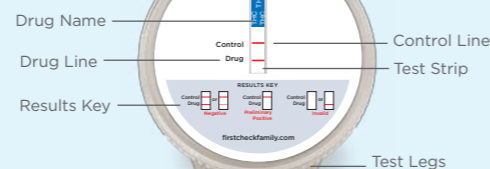

## Step 2 Activate Test

- Twist the Test Lid straight and securely onto the Collection Cup. Ensure the lid is fastened tightly so the sample does not leak.

- Tip Cup on side and rest on legs on a flat surface to activate. The First Check logo should be facing up as shown.

Test Legs


### NO DRUG DETECTED

#### Negative Result

One red or pink colored line next to the word Control **AND** one red or pink colored line next to the word Drug indicates that **no marijuana (THC) was detected** at the designated cut-off level.

*The line next to the word Drug may be lighter in color or thinner than the Control Line and still constitute a **Negative Result**.*

No further testing is required.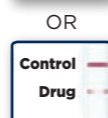

### DRUG DETECTED

#### Preliminary Positive Result


One red or pink colored line next to the word Control and **NO** line next to the word Drug indicates that **marijuana (THC) was detected** at the designated cut-off level.

Do not discard the urine sample if you receive a **Preliminary Positive Result**. We recommend sending all **Preliminary Positive Result** samples to our laboratory for free confirmatory testing. See shipping instructions.

### INVALID RESULT

If after 10 minutes **NO** line appears next to the word Control, **the result is invalid**.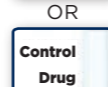

## QUESTIONS & ANSWERS

### 1. THE DRUG LINE IS LIGHTER THAN THE CONTROL LINE. DOES THIS MEAN SOME DRUG IS PRESENT?

No. Any line next to the word Drug, no matter how dark or light, means that no marijuana (THC) has been detected at the designated cut-off level, and is considered a **Negative Result**. No further testing is required.

### 2. WHAT DOES MY RESULT MEAN?

A **Negative Result** means that no marijuana (THC) has been detected in the urine sample, or the amount of marijuana (THC) present is below the cut-off level.

A **Preliminary Positive Result** means that marijuana (THC) has been detected in the home screening portion of the test. To confirm the result, we recommend sending all **Preliminary Positive Results** to our laboratory for free confidential confirmatory testing. Our laboratory uses gas chromatography/mass spectrometry (GC/MS), the gold standard in laboratory testing for most drugs. If the laboratory confirms a positive result, you can be assured that marijuana (THC) was present in the urine sample.

### 3. HOW ACCURATE IS THE FIRST CHECK® MARIJUANA TEST?

Clinical studies confirmed that over 99% of those individuals with marijuana (THC) present in their urine tested positive with the First Check® Marijuana Test.

### 4. WHAT CUT-OFF LEVEL DOES FIRST CHECK USE?

The Substance Abuse and Mental Health Services Administration (SAMHSA) has set "cut-off" levels when testing for certain drugs of abuse. The First Check® Marijuana Test is manufactured to conform to SAMHSA guidelines.

**SAMHSA CUT-OFF LEVEL FOR MARIJUANA (THC): 50 ng/ml**

SAMHSA cut-off levels are provided in nanograms per milliliter (ng/ml), which is how the concentration of a drug in urine is measured. A nanogram is one-billionth of a gram. The nanogram level does not tell you how much of a drug was taken, when a drug was taken, or the level of intoxication.

### 5. HOW DO I KNOW IF THE FIRST CHECK® MARIJUANA TEST WORKED PROPERLY? WHAT IF I AM UNSURE OF THE RESULT?

The test is working properly as long as a red or pink colored line appears next to the word Control. It is considered a **Negative Result** when a red or pink colored line appears next to the word Drug. It is considered a **Preliminary Positive Result** when **NO** line appears next to the word Drug.

If you are unsure about your result, please re-test using a new First Check® Marijuana Test.

### 6. HOW SOON CAN I READ MY RESULT?

You can read your result after 5 minutes as long as a red or pink colored line has appeared next to the word Control. Disregard all results after 10 minutes.

### 7. WHEN IS THE BEST TIME TO TAKE THE TEST?

Use first morning urine when possible, as it tends to be the most concentrated urine of the day, making it best for detecting any marijuana (THC) that may be in the system; however, First Check® Home Drug Tests can be used any time of day.

### 8. MY RESULT WAS NEGATIVE. CAN YOU GUARANTEE A NEGATIVE RESULT ON A DIFFERENT TEST?

The First Check® Marijuana Test conforms to SAMHSA guidelines, which set the standard for cut-off levels for workplace drug testing. We cannot guarantee a negative result on another test because each result can vary due to:

- **The cut-off level used** - other tests may not use the SAMHSA 50 ng/ml cut-off level.
- **The way marijuana (THC) is metabolized** - marijuana clears the body at different intervals.
- **Excessive fluid intake prior to testing** - could dilute the marijuana metabolite in the urine sample, making it more difficult to detect.
- **Timing of test** - when marijuana (THC) was used in relation to testing.

### 9. CAN SECOND-HAND SMOKE CAUSE A PRELIMINARY POSITIVE RESULT FOR MARIJUANA?

While it may be possible for THC from second-hand smoke to enter the body, it is highly unlikely to cause a **Preliminary Positive Result** at the 50 ng/ml cut-off level.

### 10. ARE THERE ANY FACTORS THAT CAN AFFECT THE TEST RESULT?

- Certain over-the-counter medications or prescription drugs may cross-react with the First Check® Marijuana Test and cause a **Preliminary Positive Result**. For a complete list of substances that may cross-react with this test, please log on to our website at [www.firstcheckfamily.com](http://www.firstcheckfamily.com).
- The test will only give accurate results on fresh human urine samples. Old or diluted urine samples may not be suitable for testing.
- If you are testing someone else, keep in mind that First Check® Home Drug Tests are only as accurate as the urine sample being tested. Samples can easily be "adulterated" (contaminated or tampered with) using common household products such as bleach and other liquids. It is best that you closely supervise the entire process.
- The First Check® Marijuana Test provides a screening result only. It is not designed to determine the actual concentration of a drug or the level of intoxication, nor is it to be used for legal purposes.

## SHIPPING YOUR FIRST CHECK® SAMPLE TO THE LAB

### Step 1 Attach Identification Label

We recommend that all **Preliminary Positive Results** be sent to our laboratory for free confidential confirmatory testing. Gas chromatography/mass spectrometry (GC/MS) is considered the gold standard in laboratory testing for most drugs.

- Make sure urine is above the Minimum Urine Level line on the cup, and that the cap is securely fastened.
- Attach the top portion of the Identification Label (with bar code) to the Collection Cup and place a check mark in the box for THC.
- Attach the lower portion of the Identification Label to the designated location on this sheet and store in a safe place.

**IMPORTANT: Without this Identification Label, we will be unable to test your sample. You will need this number to retrieve your results.**

### Step 2 Place Test in Box

- Make sure the Test Lid is securely fastened to the Collection Cup and then place it in the Plastic Transportation Bag.
- Seal bag and place into the Pre-addressed Mailing Box.

### Step 3 Ship Sample

- Close the Pre-addressed Mailing Box and secure it with packing tape.
- Samples should be mailed as soon as possible after collection.

**IMPORTANT: The mailing box is not pre-paid. To ensure prompt delivery, be sure to pay the appropriate shipping charges to the mail service you have decided to use.**

## RETRIEVING YOUR CONFIDENTIAL LABORATORY RESULT

Results will be ready 5 to 7 days after the sample is received in our laboratory.

- Dial 1-888-788-5716 or log onto [www.firstcheckfamily.com](http://www.firstcheckfamily.com).
- You can receive your results 24 hours a day, 7 days a week.

**IMPORTANT: You will be asked for the Identification Number you placed on this insert, so have it ready when you call or log on. All calls will remain strictly confidential.**

## HELP & SUPPORT INFORMATION

**Questions?** Call our toll-free Help Line at 1-888-788-5716.

Monday - Friday 8:30 a.m. - 5:00 p.m. Eastern Time.

Our professional staff is available to assist you. Visit us online at [www.firstcheckfamily.com](http://www.firstcheckfamily.com)

For in vitro diagnostic use. For external use only. Store at 40°F - 86°F (4°C - 30°C). Do not freeze. Manufactured for First Check Diagnostics, LLC Waltham, MA 02453. \*The Substance Abuse and Mental Health Services Administration (SAMHSA), has set "cut-off" levels when testing for marijuana, cocaine, methamphetamine, amphetamines, ecstasy, opiates (heroin), and PCP. The First Check® Marijuana Test is manufactured to conform to those standards. Screening tests may not detect amounts of drugs in a urine sample that are below cut-off levels. Even though some level of drug may be present in a urine sample, the sample would still be considered a **Negative Result** if the drug level is below the cut-off level.

© 2013 Alere. All rights reserved. The Alere Logo, Alere and First Check are trademarks of the Alere group of companies. Made in U.S.A.

Here are a few helpful resources on drug abuse prevention and recovery programs you may find useful.

## QUESTIONS?

Call our toll-free Help Line at 1-888-788-5716 Monday - Friday 8:30 a.m. - 5:00 p.m. Eastern Time. Our professional staff is available to assist you. You can also visit us online at [www.firstcheckfamily.com](http://www.firstcheckfamily.com)

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P/N 002645-11 8/13

The following is a list of reactive and non-reactive drugs for the First Check Marijuana Home Test, First Check 2-Drug Home Test, and First Check 4-Drug Home Test:

COMPOUND (Trade Name)	COMPOUND (Generic Name)	RESULTS
Acephen	Acetaminophen (See also Paracetamol)	Non-reactive
Aceta	Acetaminophen (See also Paracetamol)	Non-reactive
Apacet	Acetaminophen (See also Paracetamol)	Non-reactive
Dapacen	Acetaminophen (See also Paracetamol)	Non-reactive
Excedrin (combination)	Acetaminophen (See also Paracetamol)	Non-reactive
Feverall	Acetaminophen (See also Paracetamol)	Non-reactive
Panadol	Acetaminophen (See also Paracetamol)	Non-reactive
Tempra	Acetaminophen (See also Paracetamol)	Non-reactive
Tylenol	Acetaminophen (See also Paracetamol)	Non-reactive
Tylenol 3	Acetaminophen with Codeine (see also Paracetamol with codeine)	Positive for Opiates (OPI/MOR)
Tylenol with codeine	Acetaminophen with Codeine (see also Paracetamol with codeine)	Positive for Opiates (OPI/MOR)
Anadin	Acetylsalicylic acid (aspirin)	Non-reactive
Anasin	Acetylsalicylic acid (aspirin)	Non-reactive
Aspirin	Acetylsalicylic acid	Non-reactive
Bufferin	Acetylsalicylic acid (aspirin)	Non-reactive
Disprin	Acetylsalicylic acid (aspirin)	Non-reactive
Ecotrin	Acetylsalicylic acid (aspirin)	Non-reactive
Empirin	Acetylsalicylic acid (aspirin)	Non-reactive
Excedrin (combination)	Acetylsalicylic acid (aspirin)	Non-reactive
Angettes	Acetylsalicylic acid (aspirin)	Non-reactive
ASA	Acetylsalicylic acid (aspirin)	Non-reactive
Asasantin	Acetylsalicylic acid (aspirin)	Non-reactive
Baver Aspirin	Acetylsalicylic acid (aspirin)	Non-reactive
Caprin	Acetylsalicylic acid (aspirin)	Non-reactive
Excedrin	Acetylsalicylic acid (aspirin)	Non-reactive
No known trade names	Allobarbitol	Positive for Barbiturates (BAR)
Xanax	Alprazolam	Positive for Benzodiazepines (BZO)
Tryptizol	Amitriptyline	Positive for Tricyclic Antidepressants (TCA)
Elavil	Amitriptyline	Positive for Tricyclic Antidepressants (TCA)
Lentizol	Amitriptyline	Positive for Tricyclic Antidepressants (TCA)
Triptafen	Amitriptyline	Positive for Tricyclic Antidepressants (TCA)
Triptafen-M	Amitriptyline	Positive for Tricyclic Antidepressants (TCA)
Amytal	Amobarbital	Positive for Barbiturates (BAR)
Tuinal	Amobarbital	Positive for Barbiturates (BAR)
Penbritin	Ampicillin	Non-reactive
Polycillin	Ampicillin	Non-reactive
Principen	Ampicillin	Non-reactive
No known trade names	Aprobarbital	Positive for Barbiturates (BAR)
No known trade names	Barbital	Positive for Barbiturates (BAR)
Lexotan	Bromazepam	Positive for Benzodiazepines (BZO)
Butisol	Butabarbital	Positive for Barbiturates (BAR)
Soneryl	Butabarbital	Positive for Barbiturates (BAR)
Fioricet	Butalbital	Positive for Barbiturates (BAR)
Fiorinal	Butalbital	Positive for Barbiturates (BAR)
No known trade names	Butethal	Positive for Barbiturates (BAR)
Galpseud	Chlorpheniramine Maleate	Non-reactive
Haymine	Chlorpheniramine Maleate	Non-reactive
Piriton	Chlorpheniramine Maleate	Non-reactive
Frisium	Clobazam	Positive for Benzodiazepines (BZO)
Clonopin	Clonazepam	Positive for Benzodiazepines (BZO)
Klonopin	Clonazepam	Positive for Benzodiazepines (BZO)

COMPOUND (Trade Name)	COMPOUND (Generic Name)	RESULTS
Rivotril	Clonazepam	Positive for Benzodiazepines (BZO)
Codafen Continus	Codeine Phosphate	Positive for Opiates (OPI/MOR)
Codeine Linctus	Codeine Phosphate	Positive for Opiates (OPI/MOR)
Galcodine	Codeine Phosphate	Positive for Opiates (OPI/MOR)
Kapake	Codeine Phosphate	Positive for Opiates (OPI/MOR)
Migraleve	Codeine Phosphate	Positive for Opiates (OPI/MOR)
Pediatric BP	Codeine Phosphate	Positive for Opiates (OPI/MOR)
Soloadol	Codeine Phosphate	Positive for Opiates (OPI/MOR)
Tylenol	Codeine Phosphate	Positive for Opiates (OPI/MOR)
Diazemuls	Diazepam	Positive for Benzodiazepines (BZO)
Stesolid	Diazepam	Positive for Benzodiazepines (BZO)
Valclair	Diazepam	Positive for Benzodiazepines (BZO)
Valium	Diazepam	Positive for Benzodiazepines (BZO)
Benadryl	Diphenhydramine	Non-reactive
Medinex	Diphenhydramine	Non-reactive
Nytol	Diphenhydramine	Non-reactive
Panadol Night	Diphenhydramine	Non-reactive
Desoxy	d-Methamphetamine HCL	Positive for Methamphetamine (mAMP)
Methamprax	d-Methamphetamine HCL	Positive for Methamphetamine (mAMP)
Methedrine	d-Methamphetamine HCL	Positive for Methamphetamine (mAMP)
Sinequan	Doxepin	Positive for Tricyclic Antidepressants (TCA)
Xepin	Doxepin	Positive for Tricyclic Antidepressants (TCA)
Ilosone	Erythromycin	Non-reactive
Benzvancin	Erythromycin	Non-reactive
Arpimycin	Erythromycin	Non-reactive
Erycane	Erythromycin	Non-reactive
Erymax	Erythromycin	Non-reactive
Erythrocin	Erythromycin	Non-reactive
Erythroped	Erythromycin	Non-reactive
Isotrexin	Erythromycin	Non-reactive
Lactobinate	Erythromycin	Non-reactive
Stiemycin	Erythromycin	Non-reactive
Tilorvth	Erythromycin	Non-reactive
Zineryl	Erythromycin	Non-reactive
ProSom	Estazolam	Positive for Benzodiazepines (BZO)
No known trade names	Ethylmorphine	Positive for Opiates (OPI/MOR)
Rohypnol	Flunitrazepam	Positive for Benzodiazepines (BZO)
Loratab	Hydrocodone	Positive for Opiates (OPI/MOR)
Lorel	Hydrocodone	Positive for Opiates (OPI/MOR)
Vicodin	Hydrocodone	Positive for Opiates (OPI/MOR)
Dilaudid	Hydromorphone	Positive for Opiates (OPI/MOR)
Hydrostat	Hydromorphone	Positive for Opiates (OPI/MOR)
Ibugel	Ibuprofen	Non-reactive
Brufen	Ibuprofen	Non-reactive
Codafen	Ibuprofen	Non-reactive
Fenbid	Ibuprofen	Non-reactive
Ibuspray	Ibuprofen	Non-reactive
Motrin	Ibuprofen	Non-reactive
Proflex	Ibuprofen	Non-reactive
Vick's Inhaler	l-Methamphetamine HCL	May be positive for Methamphetamine (mAMP) at high concentrations
Ativan	Lorazepam	Positive for Benzodiazepines (BZO)
Anxitol	Medazepam	Positive for Benzodiazepines (BZO)
Lerisum	Medazepam	Positive for Benzodiazepines (BZO)
Medacepan	Medazepam	Positive for Benzodiazepines (BZO)

COMPOUND (Trade Name)	COMPOUND (Generic Name)	RESULTS
Nobritol	Medazepam	Positive for Benzodiazepines (BZO)
Nobrium	Medazepam	Positive for Benzodiazepines (BZO)
Doloohine	Methadone Hydrochloride	Positive for Methadone (MTD)
Methadose	Methadone Hydrochloride	Positive for Methadone (MTD)
Physetone	Methadone Hydrochloride	Positive for Methadone (MTD)
Eve (slang)	Methylenedioxyamphetamine (MDA)	Positive for Amphetamine (AMP) and Ecstasy (MDMA)
Love Drug (slang)	Methylenedioxyamphetamine (MDA)	Positive for Amphetamine (AMP) and Ecstasy (MDMA)
Adam (slang)	Methylenedioxymethamphetamine (MDMA)	Positive for Methamphetamine (mAMP) and Ecstasy (MDMA)
E (slang)	Methylenedioxymethamphetamine (MDMA)	Positive for Methamphetamine (mAMP) and Ecstasy (MDMA)
Ecstasy (slang)	Methylenedioxymethamphetamine (MDMA)	Positive for Methamphetamine (mAMP) and Ecstasy (MDMA)
XTC (slang)	Methylenedioxymethamphetamine (MDMA)	Positive for Methamphetamine (mAMP) and Ecstasy (MDMA)
Astramorph	Morphine	Positive for Opiates (OPI/MOR)
Cyclimorph	Morphine	Positive for Opiates (OPI/MOR)
Duramorph	Morphine	Positive for Opiates (OPI/MOR)
Morcap	Morphine	Positive for Opiates (OPI/MOR)
Morphine Sulfate	Morphine	Positive for Opiates (OPI/MOR)
MS Conlin	Morphine	Positive for Opiates (OPI/MOR)
Oramorph	Morphine	Positive for Opiates (OPI/MOR)
Roxanol	Morphine	Positive for Opiates (OPI/MOR)
Severedol	Morphine	Positive for Opiates (OPI/MOR)
Aleve	Naproxen	Non-reactive
Condrotec	Naproxen	Non-reactive
Napratec	Naproxen	Non-reactive
Naprosen	Naproxen	Non-reactive
Naprosyn	Naproxen	Non-reactive
Nycopren	Naproxen	Non-reactive
Synflex	Naproxen	Non-reactive
Nicoderm	Nicotine	Non-reactive
Nicorette	Nicotine	Non-reactive
Nicotinell	Nicotine	Non-reactive
Nicotrol	Nicotine	Non-reactive
Niquitin	Nicotine	Non-reactive
Mogadon	Nitrazepam	Positive for Benzodiazepines (BZO)
Somnite	Nitrazepam	Positive for Benzodiazepines (BZO)
Ox-pam	Oxazepam	Positive for Benzodiazepines (BZO)
Serax	Oxazepam	Positive for Benzodiazepines (BZO)
Fortagesic	Paracetamol (Acetaminophen)	Non-reactive
Paradote	Paracetamol (Acetaminophen)	Non-reactive
Co-codamol	Paracetamol (Acetaminophen) / Codeine Preparations	Positive for Opiates (OPI/MOR)
Codafen	Paracetamol (Acetaminophen) / Codeine Preparations	Positive for Opiates (OPI/MOR)
Co-dydramol	Paracetamol (Acetaminophen) / Codeine Preparations	Positive for Opiates (OPI/MOR)
Kapake	Paracetamol (Acetaminophen) / Codeine Preparations	Positive for Opiates (OPI/MOR)
Remedine	Paracetamol (Acetaminophen) / Codeine Preparations	Positive for Opiates (OPI/MOR)
Solpadol	Paracetamol (Acetaminophen) / Codeine Preparations	Positive for Opiates (OPI/MOR)

<b>COMPOUND (Trade Name)</b>	<b>COMPOUND (Generic Name)</b>	<b>RESULTS</b>
Tylenol 3	Paracetamol (Acetaminophen) / Codeine Preparations	Positive for Opiates (OPI/MOR)
Tvlex	Paracetamol (Acetaminophen) / Codeine Preparations	Positive for Opiates (OPI/MOR)
Combicillin	Penicillin	Non-reactive
Mefoxin	Penicillin	Non-reactive
Nembutal	Pentobarbital	Positive for Barbiturates (BAR)
Donnatal	Phenobarbital	Positive for Barbiturates (BAR)
Luminal	Phenobarbitone (see also Phenobarbital)	Positive for Barbiturates (BAR)
Centrax	Prazepam	Positive for Benzodiazepines (BZO)
Demetrin	Prazepam	Positive for Benzodiazepines (BZO)
Novocain	Procaine	Non-reactive
Afrinol	Pseudoephedrine	Non-reactive
Sudafed	Pseudoephedrine	Non-reactive
Tylenol Cold (combination)	Pseudoephedrine	Non-reactive
Pylorid	Ranitidine	Non-reactive
Zantac	Ranitidine	Non-reactive
Seconal	Secobarbital	Positive for Barbiturates (BAR)
Restoril	Temazepam	Positive for Benzodiazepines (BZO)
Halcion	Triazolam	Positive for Benzodiazepines (BZO)